

A Treatment Guide for lips, wrinkles and facial contours







What happens to the skin as we age?

- Young skin is smooth and elastic and contains a lot of hyaluronic acid
- As we grow older, the skin's capacity to bind water is reduced
- This leads to dryer, thinner and more wrinkled skin

What is hyaluronic acid?

- Hyaluronic acid is a sugar molecule that occurs naturally in the body
- In the skin hyaluronic acid attracts water, helps to hydrate the skin and gives volume and structure





Can wrinkles be treated?

 Lines, wrinkles and folds can be smoothed out by using Restylane[®]

What about the lips?

 Restylane Lipp[™] can be used to define the lip line or to add volume with a natural result

And my facial contours?

 With Restylane you can add volume to shape the contours of the face, such as the cheek and the chin





I have noticed a loss of skin elasticity and vitality in my face, can this be treated?

• Restylane Vital™ rejuvenates the skin. It restores the hydrobalance and improves elasticity and tone, thereby creating a fresh and natural look

Can something be done with the back of my hands or the décolletage?

 Restylane Vital can be used for the face, neck, décolletage and the back of the hands. Restylane Vital will make the skin soft and smooth



What is Restylane®?

- Restylane is a crystal clear gel based on non-animal hyaluronic acid. Restylane is produced by the Swedish biotechnology company Q-Med AB
- The Restylane products are biodegradable, which means that when the gel is broken down it disappears unnoticed from the body





Why Restylane®?

- More than 10 million treatments have been performed worldwide since 1996
- Restylane has through clinical evidence been proven to be safe, effective and long lasting. Q-Med AB continues to invest in research and development in the interests of further improving safety and efficacy
- Restylane treatment provides an easy way to stay in control of your looks. It is a quick procedure, with minimal associated downtime. The results of a single treatment can be seen instantly




Are there different Restylane® products for different needs?

 The Restylane portfolio comprises a range of products for facial beauty treatments, designed specifically to reduce the appearance of lines, folds and wrinkles, to contour and create volume, and to rejuvenate the skin

Are there any side-effects?

- After the injection of Restylane, some common injectionrelated reactions might occur (Reactions include: swelling, redness, pain, itching, discoloration and tenderness)
- These reactions typically resolve spontaneously
- Other type of local reactions occur but are rare

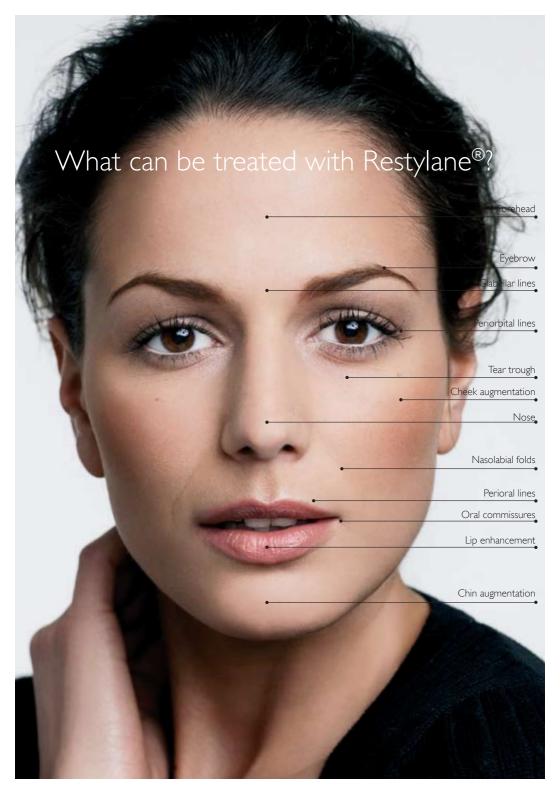






How is a wrinkle treatment done?

- The crystal-clear gel is injected into the skin in tiny amounts
- This creates volume under the wrinkle that is lifted up and smoothed out

How is a lip treatment done?

• The gel can be injected along the lip line to emphasize the contour or directly into the upper and/or lower lip for more volume

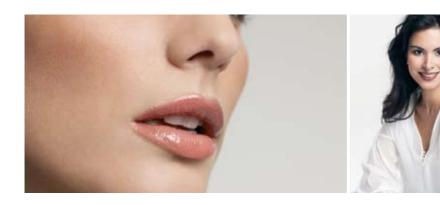

How long does a Restylane® treatment take?

- A treatment session takes about 30 minutes
- No prior skin test is necessary which means the treatment can be done instantly
- Results will be seen immediately

Do I need pain relief?

- Pain relief is generally not necessary but is easy to arrange.
 A topical anaesthetic is usually enough
- If the lips are treated pain relief is recommended





How long will it last?

- Restylane® products are long lasting but not permanent
- Many choose to have a follow up treatment 6 to 12 months after the initial treatment of wrinkles
- When the lips have been treated a follow-up treatment is normally needed after about 6 months

What will I look like immediately after a treatment?

- After treatment slight redness, swelling, tenderness and an itching sensation can be expected
- These symptoms are temporary and typically resolve spontaneously



When should I come back?

- To achieve the best possible result a touch-up treatment within 2-4 weeks may be necessary. This will also increase the duration of the treatment
- When a follow-up treatment is performed a smaller amount of product is normally required

Treatment results



Restylane®

Treatment results



Treatment results





Natural result
Safe products
Easy treatment

www.restylane.com

)6. Restylane and all other product names in the Restylane family, as well as NASHA, are trademarks of Q-Med AB.